

```

Nucleus = "H1";
SmartSelect = "yes";
Coil 1 (exclude) = "None";
Uniformity = "CLEAR";
FOV      FH (mm) = 256;
         AP (mm) = 256;
         RL (mm) = 176;
ACQ voxel size  FH (mm) = 1;
                 AP (mm) = 1;
                 RL (mm) = 1;
Recon voxel size FH (mm) = 1;
                 AP (mm) = 1;
                 RL (mm) = 1;
Fold-over suppression = "no";
Slice oversampling = "user defined";
oversample factor = 1;
RF select. FOS = "no";
Reconstruction matrix = 256;
SENSE = "yes";
P reduction (AP) = 3;
S reduction (RL) = 1;
CS-SENSE = "no";
k-t BLAST = "no";
Stacks = 1;
slices = 176;
slice orientation = "sagittal";
fold-over direction = "AP";
fat shift direction = "F";
Stack Offc. AP (P=+mm) = -7.9176302;
              RL (L=+mm) = -1.58789158;
              FH (H=+mm) = -160;
Ang. AP (deg) = 1.97074461;
      RL (deg) = -0.0162523538;
      FH (deg) = 0.529203415;
Free rotatable = "no";
Multi-chunk = "no";
O-MAR = "no";
Large table movement = "no";
PlanAlign = "no";
REST slabs = 0;
Catheter tracking = "no";
Interactive positioning = "no";
Allow table movement = "no";
Patient position = "head first";
Patient body position = "head first";
Patient orientation = "supine";
Patient body orientation = "supine";
Scan type = "Imaging";
Scan mode = "3D";
technique = "IR";
Acquisition mode = "cartesian";
Fast Imaging mode = "TSE";
3D VIEW = "Brain FLAIR";
shot mode = "multishot";
TSE factor = 150;
startup echoes = 6;
profile order = "linear";
turbo direction = "Y";
DRIVE = "no";
fid reduction = "default";
Echoes = 1;
partial echo = "no";
TE = "user defined";
(ms) = 271;
Refocusing control = "constant";
angle (deg) = 40;
bright fat reduction = "no";

```

```

TR = "user defined";
  (ms) = 4800;
Halfscan = "yes";
  factor Y = 0.899999976;
  factor Z = 1;
Water-fat shift = "user defined";
  (pixels) = 0.699999988;
IR delay (ms) = 1650;
  +multiple TI = "no";
  dual = "no";
  power = "1";
RF Shims = "fixed";
Shim = "auto";
mDIXON = "no";
Fat suppression = "SPIR";
  strength = "strong";
  frequency offset = "default";
Grad Rev Fat suppr = "no";
Water suppression = "no";
BB pulse = "no";
MTC = "no";
+FLEX = "no";
T2prep = "yes";
  +Phase cycling = "default";
  +TRUST = "no";
  echo time (ms) = 125;
  refocusing pulses = 4;
  +CSC T2prep adiabatic = "yes";
  +set RF off = "no";
Research prepulse = "no";
MDME = "no";
Zoom imaging = "no";
Diffusion mode = "no";
T1 mapping = "no";
Multi-transmit = "yes";
Transmit channels = "both";
SAR mode = "high";
B1 mode = "default";
SAR allow first level = "yes";
PNS mode = "high";
Gradient mode = "maximum";
SoftTone mode = "no";
+user options = 0;
Cardiac synchronization = "no";
Heart rate > 250 bpm = "no";
Respiratory compensation = "no";
Navigator respiratory comp = "no";
Flow compensation = "no";
Motion smoothing = "no";
NSA = 1;
CENTRA = "no";
Manual start = "no";
Dynamic study = "no";
Arterial Spin labeling = "no";
  +Mag reset = "no";
Preparation phases = "auto";
  +Power optimization = "auto";
Interactive F0 = "no";
SmartPlan survey = "no";
B0 field map = "no";
B1 field map = "no";
MIP/MPR = "MPR";
  protocol = "compose";
Images = "M", (3) "no";
Autoview image = "M";
Reference tissue = "Grey matter";
Recon compression = "No";

```

```
Preset window contrast = "soft";
Reconstruction mode = "real time";
Save raw data = "no";
Hardcopy protocol = "no";
Image filter = "system default";
Uniformity correction = "no";
Geometry correction = "default";
Elliptical k-space shutter = "default";
IF_info_seperator = 1634755923;
Total scan duration = "05:55.2";
Rel. SNR = 1;
Act. TR/TI (ms) = "4800 / 1650";
Act. TE (ms) = "271";
ACQ matrix M x P = "256 x 256";
ACQ voxel MPS (mm) = "1.00 / 1.00 / 1.00";
REC voxel MPS (mm) = "1.00 / 1.00 / 1.00";
Scan percentage (%) = 100;
Act. slice gap (mm) = 0;
WFS (pix) / BW (Hz) = "0.658 / 659.8";
TSE es / shot (ms) = "3.6 / 569";
TEeff / TEequiv (ms) = "271 / 120 ";
Min. TR/TI (ms) = "2367 / 50";
Local torso SAR = "< 15 %";
Whole body SAR / level = "< 0.4 W/kg / normal";
SED = "< 0.1 kJ/kg";
Coil Power = "14 %";
Max B1+rms = "0.97 uT";
PNS / level = "95 % / 1st level";
dB/dt = "102.1 T/s";
Sound Pressure Level (dB) = 22.9136696;
```