

# SIEMENS MAGNETOM TrioTim syngo MR B17

\\USER\JPHILLIPS\TECHDEV\_07193\andy\_spiral\_pcas\dti\_45dir\_1000\_AP  
 TA: 0:59    PAT: 2    Voxel size: 2.0x2.0x2.0 mm    Rel. SNR: 1.00    SIEMENS: ep2d\_diff

## Properties

Prio Recon	Off
Before measurement	
After measurement	
Load to viewer	On
Inline movie	Off
Auto store images	On
Load to stamp segments	Off
Load images to graphic segments	Off
Auto open inline display	Off
Start measurement without further preparation	On
Wait for user to start	Off
Start measurements	single

## Routine

Slice group 1	
Slices	80
Dist. factor	0 %
Position	Isocenter
Orientation	Transversal
Phase enc. dir.	P >> A
Rotation	180.00 deg
Phase oversampling	0 %
FoV read	256 mm
FoV phase	100.0 %
Slice thickness	2.0 mm
TR	9800 ms
TE	84 ms
Averages	1
Concatenations	1
Filter	Prescan Normalize
Coil elements	HEA;HEP

## Contrast

MTC	Off
Magn. preparation	None
Fat suppr.	Fat sat.
Averaging mode	Long term
Reconstruction	Magnitude
Delay in TR	0 ms
Multiple series	Off

## Resolution

Base resolution	128
Phase resolution	100 %
Phase partial Fourier	6/8
Interpolation	Off
PAT mode	GRAPPA
Accel. factor PE	2
Ref. lines PE	24
Matrix Coil Mode	Auto (Triple)
Reference scan mode	Separate
Distortion Corr.	Off
Prescan Normalize	On
Raw filter	On
Elliptical filter	Off
Hamming	Off

## Geometry

Multi-slice mode	Interleaved
Series	Interleaved

Special sat.                      None

## System

Body	Off
HEP	On
HEA	On
Positioning mode	REF
Table position	H
Table position	0 mm
MSMA	S - C - T
Sagittal	R >> L
Coronal	A >> P
Transversal	F >> H
Coil Combine Mode	Adaptive Combine
Auto Coil Select	Default

Shim mode	Standard
Adjust with body coil	Off
Confirm freq. adjustment	Off
Assume Silicone	Off
? Ref. amplitude 1H	0.000 V
Adjustment Tolerance	Auto
Adjust volume	
Position	Isocenter
Orientation	Transversal
Rotation	180.00 deg
R >> L	256 mm
A >> P	256 mm
F >> H	160 mm

## Physio

1st Signal/Mode	None
Resp. control	Off

## Diff

Diffusion mode	Free
Diff. weightings	1
b-value	1000 s/mm <sup>2</sup>
Diff. weighted images	On
Trace weighted images	Off
Average ADC maps	Off
Individual ADC maps	Off
FA maps	Off
Mosaic	On
Tensor	Off
Noise level	40
Diff. directions	3

## Sequence

Introduction	Off
Bandwidth	1628 Hz/Px
Free echo spacing	Off
Echo spacing	0.7 ms
EPI factor	128
RF pulse type	Normal
Gradient mode	Fast